shower chair
with back

silla con respaldo
para la ducha

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Item # RTL12202KDR

EU Authorized Representative
Y Sung Handelsvertretung
Duesselthaler Str. 24 40211
Duesseldorf, Germany

www.drivemedical.com

www.drivemedical.com

driveTM

driveTM

assembly instructions

Weight Capacity: 300 lbs (136 Kg)

1. Remove contents from carton.
2. Place the leg section marked with a “B” into the groove of the bottom of the seat marked “B”. Place the leg section marked with an “A” on top of the “B” tube and into the groove of the seat marked “A”. Insert the shorter thumb screw through the middle hole but do not fully tighten the screw.

3. Assemble backrest by taking the “L” shaped backrest supports and securing them to the back rest using the shorter thumb screws as shown below.

4. Attach backrest to the seat by aligning holes on “L” shaped frame supports with the holes on the leg section. Secure the legs of the frame marked “B” in place by using the longer thumbs screws by making sure to pay attention to the “rear” of the seat marked on the bottom.

Note: Seat depth can be adjusted by choosing 1 of the 2 holes on the frame supports.

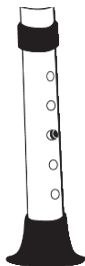
5. Secure the legs of the frame marked “A” in place with the remaining 2 longer thumb screws.

6. Fully tighten the middle shorter middle screw and 2 screws in the B frame and check to make sure all screws are fully tightened.
7. Insert legs into leg frame by depressing push pins, sliding legs over the receptacle and setting to desired height. Make sure all the legs are set to the same height.

1. Before use, adjust height of each leg to your specific needs, making sure all legs are at the same height. When desired height is reached, lock legs into place by aligning buttons into proper holes, per illustration.
2. To be sure legs are locked into place by pushing down on or by pulling legs.

additional precautions

1. Inspect all parts for shipping damage before assembly. If there is shipping damage - DO NOT USE. Contact dealer/carrier for further instructions.
2. Make certain that the height adjusting snap buttons fully protrude through the same respective hole of each leg extension. This will ensure that the leg extensions are securely locked in position and an even height is achieved.
3. Regularly check thumb screws to make sure they are fully tightened.
4. Check the rubber tips on the leg extensions for rips, wear or if they are missing. Immediately replace any or all if any of these imperfections exist.
5. All four leg extensions with rubber tips MUST touch the floor simultaneously at all times.
6. When used in a tub, make certain the rubber tips are not touching the sides of the tub wall.
7. DO NOT stand on chair.

NOTE: To maintain the beauty of the original finish, clean with mild soap and rinse with warm water. Wipe bath seat dry after each use.

maintenance

Drive recommends periodic visual inspection of this product by the user, to ensure that all parts and hardware are secure, that components are in good working order and not worn, torn, frayed, loose, or rusted and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or compromised, do not use the product. If you experience problems, please contact the provider that sold you the product or visit www.drivemedical.com/help for more info.

lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

© 2020 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in China

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© 2020 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.