



Application Instructions

Product: Universal Wrist Brace

Model: 340-R / 350-L

Indicated when wrist immobilization and full hand function are desired

Universal sizing in left and right reduces inventory

Dual lacing closure provides greater contouring and support

Contoured palmer stay is malleable and the dorsal stay can be positioned as desired

Soft and comfortable construction

Universal left and right



Sizing: Universal Left and Right

HOW TO APPLY:

- 1) Unfasten straps and slide affected hand into brace.
- 2) Position palm stay firmly against the palm of the hand.
- 3) Ensure the edge of the brace is below the knuckles.
- 4) Detach and align dorsal stay with middle knuckle. Reattach.

NOTE: Pediatric braces do not have a detachable dorsal stay

- 5) Secure the thumb-web strap.

NOTE: For braces with thumb support; wrap thumb strap around thumb and secure.

- 6) Pull and wrap the lacing straps around wrist.
- 7) Secure when the desired fit and level of compression is reached.

CARE INSTRUCTIONS:

Fasten Closures, Hand Wash Cold, Use Mild Soap Only, Air Dry Away From Heat

CAUTION:

If your pain increases or persists or you experience any unusual reactions while using this product, consult your healthcare provider immediately.