



Application Instructions

Product: Neoprene Wrist/Thumb Wrap Support
Model: 21002

- Neoprene coated with 4-way stretch nylon fabric.
- Provides compression and warmth to wrist/thumb area.
- Extended thumb provides abduction.
- Hook and loop closure for easy adjustment.
- Universal design fits a range of sizes.
- Fits either left or right wrist.



Order #	Size
21002	Universal

SIZING:

Universal design fits a range of sizes and fits left or right.

HOW TO APPLY:

Unfasten closures. Place thumb through thumb sleeve and wrap the support around the wrist. Pull tight but not too snug. Attached the hook and loop closure. Readjust if necessary.

CARE INSTRUCTIONS:

Fasten closures. Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

WARNING:

Contact neoprene material with skin may cause irritation. Persons with a known susceptibility to dermatitis or with broken skin should avoid contact with neoprene. Anyone who develops skin irritation from contact with this material should discontinue its use immediately. Should not be worn by persons with phlebitis or circulatory disorders. Not recommended to be worn while sleeping.

SUGG HCPCS CODE: A4466

Advanced Orthopaedics, Inc.
326 Main Street Harleysville, PA 19438
800.270.7074 info@advanced-orthopaedics.com