Bath Chair without Back

Chaise de bain sans dossier

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User Instructions and Warranty

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Maintenance

Periodically, inspect the bath seat for damaged or loose components, and ensure that all parts are secure.

Check that there are no obstructions to normal, safe operation.

To maintain appearance and good hygiene, clean periodically with a non-abrasive cleaner or household cleaner with warm water. Rinse with warm water and wipe dry after cleaning.

DO NOT use abrasive powders, cleaners, cloths, or scouring pads.


Wipe bath seat dry after each use.

NOTE: If any part or mechanism appears to be unsafe or damaged, **DO NOT USE THE PRODUCT**. If you experience problems, please contact Medline at **1-800-MEDLINE**.

Contents

- If exposed to cold temperatures, allow contents to come to room temperature before completing assembly. Remove contents from box.
- Ensure that all parts are present before assembling the bath seat.

Part Identification	Part Name	Quantity
A	Bath Seat	1
B	Leg Frame	2
C	Thumbscrew	5
D	Support Bar	2
E	Leg	4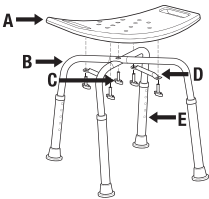

Assembly

1. Place bath seat upside down on a flat surface.
2. Place the leg frame marked with a "B" into the groove of the seat bottom marked "B" (**Figure 1**).
3. Place the leg frame marked with an "A" on top of the "B" leg frame and into the groove of the seat bottom marked "A" (**Figure 1**).
4. Insert a thumbscrew through the middle hole at the intersection of the leg frames, but do not fully tighten the screw (**Figure 1**).
5. Lay the support bars across both leg frames, parallel to the handles on the seat bench. Secure each end of the support bar to the pertinent leg frame with a thumbscrew.
NOTE: DO NOT use the bath seat without the support bars.

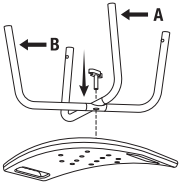
Figure 1

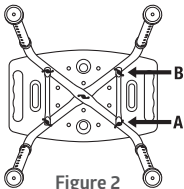
Figure 2

6. Insert each leg onto a leg frame by depressing the pushpins (**Figure 3A and 3B**). Slide the leg frames over the leg and setting to the same desired height (**Figure 3C**).

Adjusting Leg Height

1. On each leg, depress the push button and slide to desired height by aligning with a hole (**Figure 3B**). Let the push button snap into a hole to lock the leg in place (**Figure 3C**).
2. Check that the legs are locked in place by pushing down and pulling up on each leg.
3. Ensure all legs are at the same height.

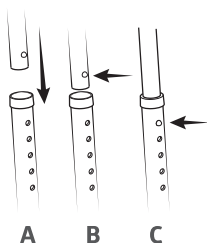
Figure 3

Cautions

- To ensure the safe assembly and operation of this product, these instructions must be followed. **FAILURE TO FOLLOW INSTRUCTIONS MAY RESULT IN SERIOUS INJURY TO THE USER OR DAMAGE TO THE PRODUCT.** If you are unable to understand these warnings, cautions and/or instructions, contact a healthcare professional, dealer or Medline before attempting to use this product.
- **DO NOT** assemble and use this product without first reading and understanding this user's guide, otherwise injury or damage may occur.
- Users with limited physical strength and/or mobility should be supervised and/or assisted while using the product. Prior to use, ensure the following:
 - Inspect all parts for possible damage before assembling. If there is any damage – **DO NOT USE**. Contact Medline at **1-800-MEDLINE**.
 - Check that the height adjusting pushbuttons fully protrude through the same corresponding hole of each leg extension. This ensures that the leg extensions are securely locked in place at an even height.
 - Make sure all screws, nuts and bolts are tightened.
 - Verify that all tips are in place on the leg extensions and not ripped, worn or missing. Replace any missing or damaged tips.
 - The tips of all four leg extensions **MUST** touch the floor simultaneously at all times.
 - Ensure the chair remains balanced. Excessive twisting or turning while seated on the chair may cause a loss of balance and/or damage to the product.
 - User evenly distributes weight between the side supports when transferring in and out of the chair.
- **DO NOT** stand on the bath seat.
- **DO NOT** exceed the user weight capacity of **400 lb (181 kg)**.
- **DO NOT** place bath seat on uneven surfaces.
- **DO NOT** use if wobbly, unstable or not level.
- **DO NOT** make any adjustments while using the product.
- **DO NOT** use product for anything other than its intended use.

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Limited Warranty

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as tip accessories, casters and grips, which are subject to normal wear and replacement.

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Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual.

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