24 – 10060 – 0

## Model C2 Backrest Mounting System

Unavailable

<https://whill.inc/>

Model: C2 series

1.6kg

The symbols used throughout this manual are described below. Be sure to observe the instructions indicated by each symbol.

| Symbol | Description                                                                                                 |
|--------|-------------------------------------------------------------------------------------------------------------|
|        | Instructs to read the instruction manual before starting work or before operating equipment or machinery.   |
|        | Indicates instructions to prevent a hazardous situation that may result in injury, property damage or both. |

| Symbol | Description                                                                                                                                                 |
|--------|-------------------------------------------------------------------------------------------------------------------------------------------------------------|
|        | Indicates a prohibited act.                                                                                                                                 |
|        | Indicates precautions or reference information that must be read in order to prevent device malfunctions or failures, and to ensure comfortable device use. |

1. This manual describes how to use the [Model C2 Backrest Mounting System] safely. It also describes how to install the product and replace parts for dealers, distributors, and sales representatives. To use this product properly, be sure to read this manual carefully before use, and contact your dealer if you have any questions.
2. Keep this manual together with the instruction manual for the Model C2.
3. Before using the product, be sure to refer to the instruction manual of the Model C2 for operating procedures and precautions. This manual contains only items related to the [Model C2 Backrest Mounting System].
4. The figures in this manual may differ in appearance from the actual product.
5. Although every effort has been made to ensure the accuracy of the contents of this manual, please contact your dealer if you have any questions, errors, or concerns.
6. A label with product information is attached to this product at the location indicated in the figure below.

7. The table below shows the specifications of the Model C2 with the [Model C2 Backrest Mounting System] installed. For other specifications, please refer to the instruction manual of Model C2.

| Item                   | Value                                              | Note                                           |
|------------------------|----------------------------------------------------|------------------------------------------------|
| Device weight          | 16 in : 51.3kg<br>18 in : 52.0kg<br>20 in : 52.2kg | Values do not include the back support weight. |
| Back support angle     | 90°                                                | -                                              |
| Device height          | 760mm – 840mm                                      | Values do not include the back support height. |
| Weight capacity        | 115kg (16" seat)<br>136kg (18" and 20" seats)      | Be sure to properly install the back support.  |
| Maximum inclined slope | 6°                                                 | Be sure to properly install the back support.  |

This product contains the following contents. Be sure to check for missing or damaged items. If any of the contents are missing or damaged, please contact your dealer.

- [Model C2 Backrest Mounting System] (1 set)
- Stoppers (1 set for left and right sides to be used for the Model C2 with the seat size of 16in)
- Stopper mounting parts: M6 bolts (2 pieces), M6 washers (2 pieces), M6 nuts (2 pieces)
- [Model C2 Backrest Mounting System] Instruction Manual (1 copy, this manual)

x 2

x 1

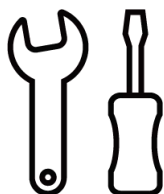
x 2


x 2

x 1

Hex wrench
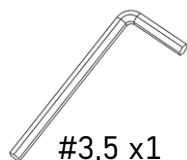
#3,5 x1

Wrench

#10,12,13 x1

Torque Wrench

#10,12,13 x1

Ruler

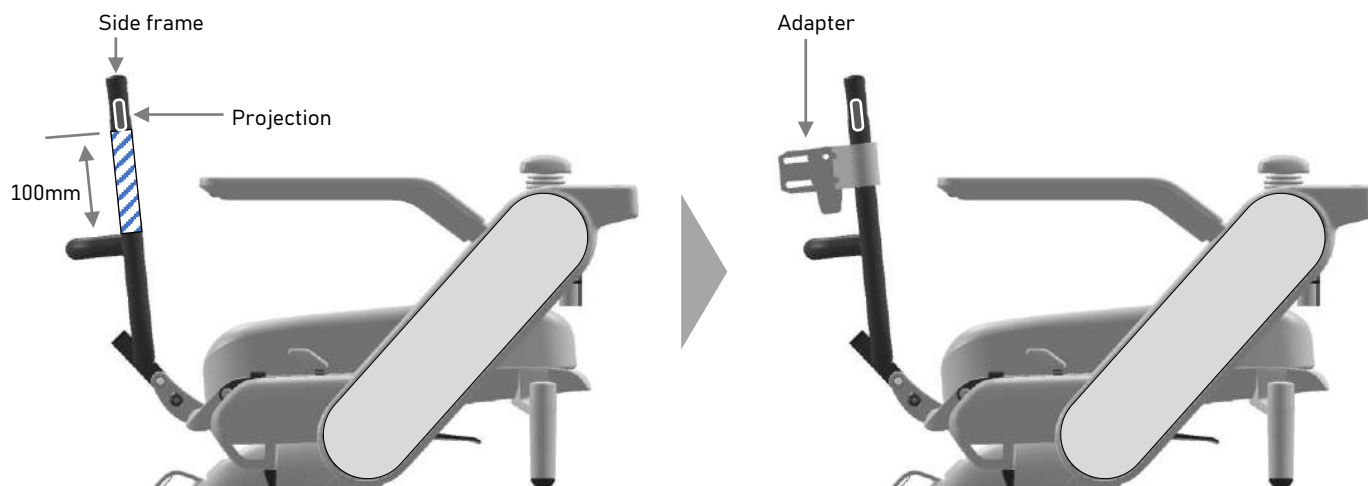
60,30cm x1

#### Installation of back support (sold separately)

- Attach the adapter supplied with the back support to the side frames of the [Model C2 Backrest Mounting System]. Adjust the mounting position so that it is within the range of approximately 100 mm above the back support frame and below the projection on the side frame. Failure to observe the mounting position may cause the back support to lose balance and fall over when traveling over bumps or hills.

For detailed installation instructions, please refer to the instruction manual that comes with the back support.

The manufacturer has verified the installation of low, mid, and tall back supports with the Varilite Icon Back System (see table below). When using back supports other than the Varilite Icon Back System, check the position and angle of the back support by referring to the "Adjusting the Back Support" section.


List of sizes of Varilite Icon Back Systems that can be installed, as confirmed by the manufacturer

| Size of Varilite Icon Back System | Model C2 Seat Size |            |            |
|-----------------------------------|--------------------|------------|------------|
|                                   | 16"                | 18"        | 20"        |
| Low                               | 35 – 45 cm         | 35 – 50 cm | 35 – 50 cm |
| Mid                               | 30 – 45 cm         | 30 – 50 cm | 30 – 55 cm |
| Tall                              | 30 – 45 cm         | 30 – 50 cm | 30 – 55 cm |



## Adjusting the Back Support

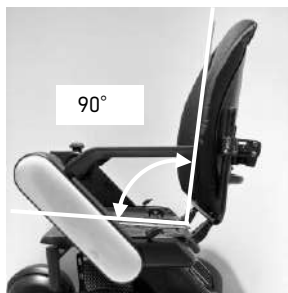
- Install the back support. If the back support angle is adjustable, adjust it so that it is perpendicular to the seat.

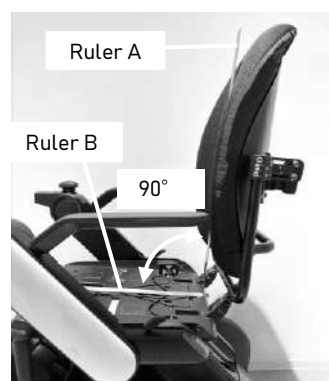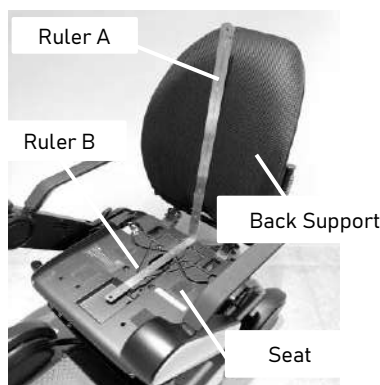- Place Ruler B on the seat with Ruler A along the center of the back of the [Model C2 Backrest Mounting System].
- Place Ruler A so that it intersects Ruler B placed on the seat at a right angle.

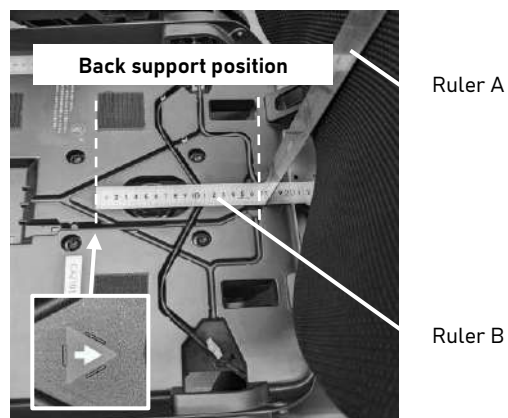- Measure the back support position by measuring the distance from the triangular guide on the seat to the intersection of Rulers A and B.

- Adjustment range of the back support



|          | Model C2 Seat Size                                      |     |     |
|----------|---------------------------------------------------------|-----|-----|
|          | 16"                                                     | 18" | 20" |
| Position | 95 – 205 mm<br>(all seat size)                          |     |     |
| Angle    | Within 10° from the upright position<br>(all seat size) |     |     |


## Installing the [Model C2 Backrest Mounting System]

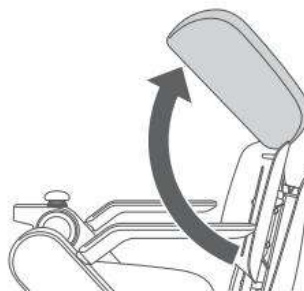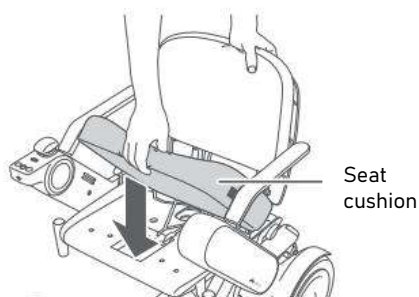- Remove the seat cushion and back support cushion. Store or dispose of the removed back support parts.


- Loosen and remove the four M6 screws securing the back support, and remove the back support.

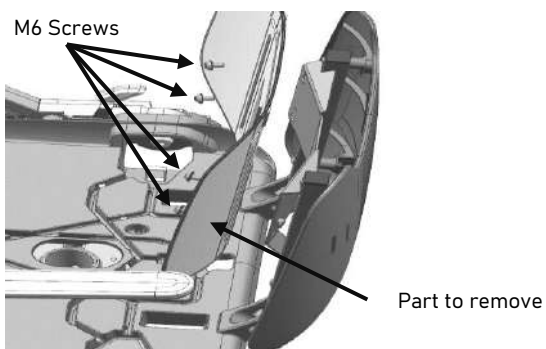Tool: Hex wrench #5

- Loosen and remove the four M5 screws securing the exterior and remove the exterior.

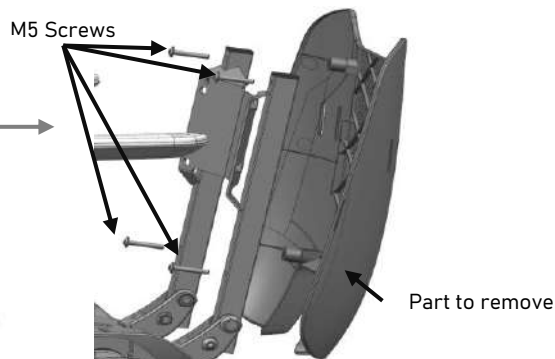Tool : Hex wrench #3


- Loosen and remove the two M6 screws, two M6 nuts, two M8 screws, and two M8 nuts from the back support frame, and remove the back support frame. There are four spacers in between, so be sure to keep them.

Tool (M6 screws): Hex wrench #5

Tool (M6 nuts): Wrench #10

Tool (M8 screws): Wrench #13

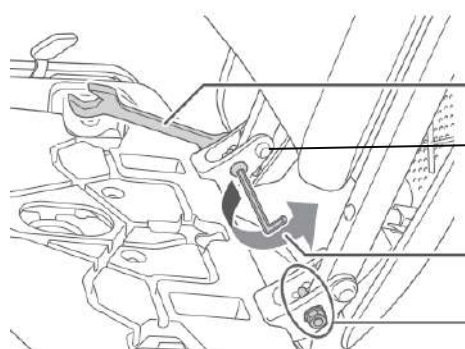Tool (M8 nuts): Wrench #12

**i** Removed parts to keep

#### Installing the [Model C2 Backrest Mounting System]

- Install and secure the [Model C2 Backrest Mounting System] using the spacers, bolts, and nuts removed in step 4. Insert the M5 bolts at the 90° position indicated on the mounting hole label. After installation, confirm that the frame is secured.

Tool (M6 screws): Hex wrench #5

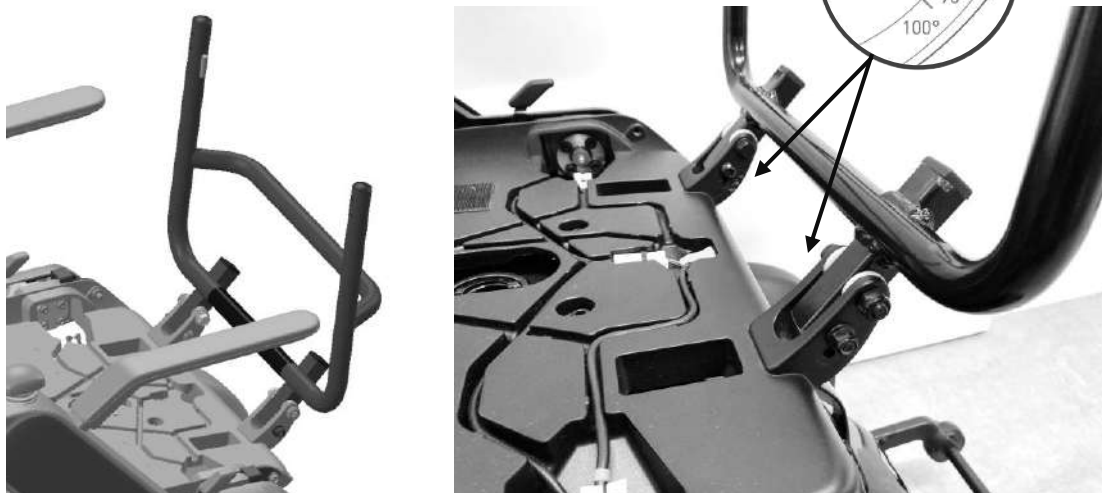
Tool (M6 nuts): Wrench #10

Tightening torque: 6Nm


Tool (M8 screws): Wrench #13

Tool (M8 nut): Wrench #12

Tightening torque: 5Nm

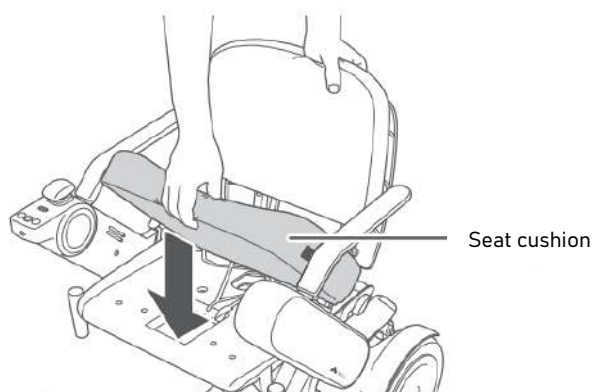- Install the seat cushion.  
If the seat size is 16in, install the stopper to prevent interference between the arm and back support (see next page).

**i** Seat assemblies with the [Model C2 Backrest Mounting System] installed cannot be packed in the Model C2 seat assembly packing box.

## Installation of stoppers (16" seat only)

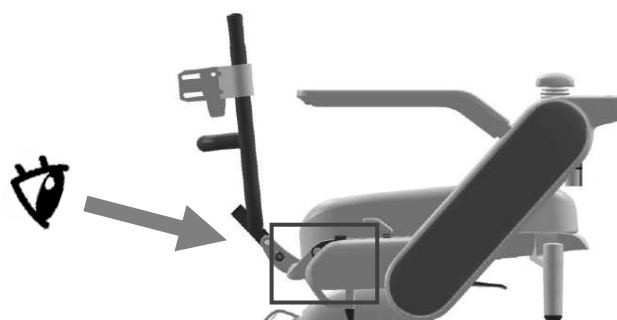
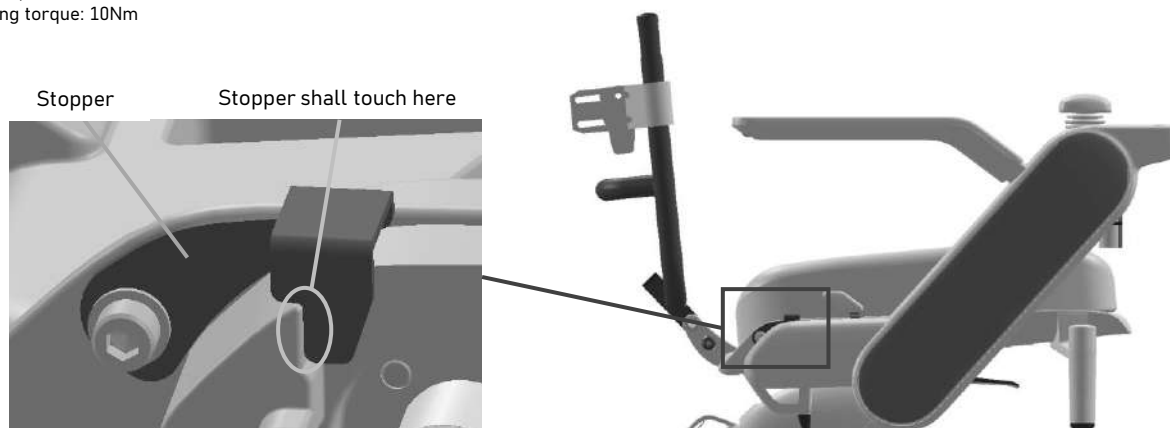
**i** The stopper shall be installed to prevent interference between the arm and back support when installing the [Model C2 Backrest Mounting System] on a 16" seat. When installing the Lap belt and/or Self standing cane holder, please read the instruction manual of each product and follow the instructions below.

- Install the parts in the order shown in the figure. Installed stopper shall touch the point shown in the figure on the right. The figure below shows the right side of the main unit, but the left side should be installed in the same manner.


Tool (Screws): Hex wrench #5

Tool (Nuts): Wrench #10


Tightening torque: 10Nm


### Install the [Model C2 Backrest Mounting System] only
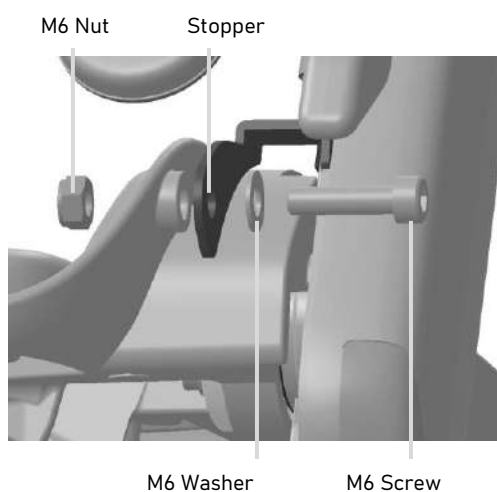
### Installing the Lap belt
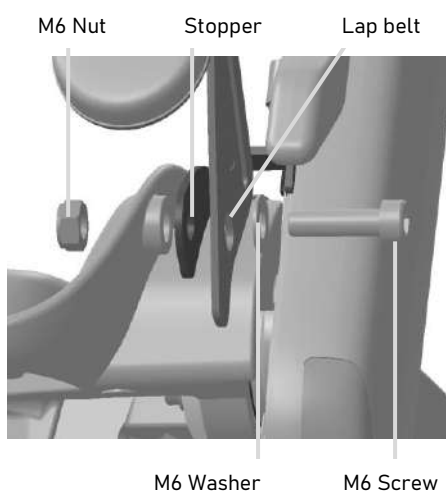
### Installing the Lap belt and Self standing cane holder

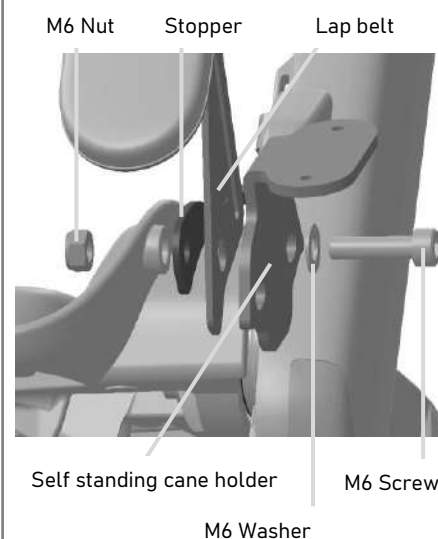Although the [Model C2 Backrest Mounting System] is designed and manufactured with due consideration for safety, improper use or failure to observe precautions may result in damage to the human body or household goods. Please read the following safety instructions carefully before using this product to ensure correct use and proper handling.

#### Installation Precautions

1. Installation of the [Model C2 Backrest Mounting System] shall only be performed by persons trained in the maintenance of the Model C2. If untrained persons perform maintenance, it may cause malfunction or damage to the product.
2. The back support shall be installed within the specified range. If installed outside the specified range, the Model C2 may lose balance and fall over when driven over bumps or hills.

#### Precautions for use

1. Do not push, pull, or lift the Model C2 with the [Model C2 Backrest Mounting System]. Doing so may cause loss of balance and an unexpected accident such as tipping over.
2. Do not hang luggage on the [Model C2 Backrest Mounting System]. It may cause loss of balance on a slope. Always use a storage basket to carry luggage.
3. When using a passenger car, train, bus, or other public transportation, do not use straps, belts, or other objects to the [Model C2 Backrest Mounting System] to secure the Model C2. Excessive load applied with the emergency brake may cause damage to the unit or the [Model C2 Backrest Mounting System], or cause it to fall over.
4. Make sure that the [Model C2 Backrest Mounting System] and the back support are correctly and firmly installed without any looseness before getting into the device. Looseness may cause an unexpected accident. Refer to the instruction manual that comes with the back support for information on installing the back support.
5. Observe the weight capacity and incline capability specified for each back support adjustment position (see table below). Exceeding the weight capacity and incline capability may result in unexpected accidents, such as a longer braking distance on downhill slopes or the main unit tipping over.

| Weight Capacity<br>(Seat size)                                                                    | Incline Capability                                                                                | Installation type of the Varilite Icon Back System<br>(Top view of the adapter, refer to the instruction manual of the back support) |  | Back support position |
|---------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------------------------------------------------------|--|-----------------------|
| 115kg (16")<br>136kg (18" and 20")                                                                | 6°                                                                                                | EXTRA DEPTH                                                                                                                          |  | 205mm                 |
|                                                                                                   |                                                                                                   | BASIC                                                                                                                                |  | 170mm - 205mm         |
|                                                                                                   |                                                                                                   | GROWTH                                                                                                                               |  | 95mm - 130mm          |
| <br>DO NOT USE | <br>DO NOT USE | EXTRA GROWTH                                                                                                                         |  | 60mm - 95mm           |

Canada, USA

us.info@whill.inc TEL: (844) 699-4455

Japan

jp.contact@whill.inc TEL: 0800-080-4338

Australia, China, Hong Kong, India, Indonesia, Korea, Malaysia, New Zealand, Philippines, Singapore, Taiwan, Thailand

apac.sales@whill.inc TEL: +86-0512-6856 2019

EU countries, Norway, Switzerland, UK, Bahrain, Egypt, Israel, Qatar, Saudi Arabia, UAE

eu.info@whill.inc

WHILL株式会社：東京都品川区東品川2-1-11  
WHILL Inc：2 - 1 - 11 Higashishinagawa, Shinagawa-ku, Tokyo 140-0002, Japan

CMC Medical Devices & Drugs S.L.  
C/ Horacio Lengo n18 C.P 29006 Málaga-Spain

\*China and Asia-Pacific countries only: For more information of local representative, please refer to the user manual of the device / \*仅限于中国及亚太地区：更多关于当地代表的信息详见设备的用户手册

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