


Control Panel Guide

Product Code- FC-PHR0025

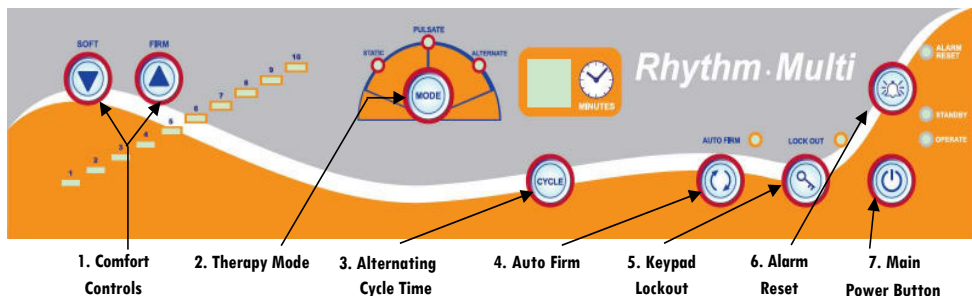

<p>Comfort Controls— Choose from 1 of 10 selectable levels of air pressure for support of the patient. Manufacturer recommends using a hand check to select proper pressure setting for patient.</p>	<p>Keypad Lockout — Press and hold for 5 seconds to lock/unlock all controls from patients or other non-caregivers</p>
<p>Therapy Mode — Pressing the MODE button will toggle between static low air loss (STATIC), alternating pressure and pulsation therapies</p>	<p>Alarm Reset — If the power failure alarm is triggered, this button will silence and reset the alarm.</p>
<p>Alternating Cycle Time — When the Therapy Mode is in the ALTERNATE setting, the CYCLE button provides a selectable alternating pressure cycle between 3 and 95 minutes</p>	<p>Main Power Button — after the main power switch is turned on (side of pump) press and hold this button to move pump from Standby Mode to Operate Mode which activates the airflow.</p>
<p>Auto Firm — Maximum inflation of the mattress to allow for easier egress, ingress or patient transfer.</p>	

Rhythm Multi Suggested Weight Guideline

Level	*Suggested weight range (KG)	*Suggested weight range (LB)
P1	40-75kgs	88-165lbs
P2	64-102kgs	141-225lbs
P3	86-125kgs	190-275lbs
P4	113-150kgs	250-330lbs
P5	136-193kgs	300-425lbs
P6	179-240kgs	395-530lbs
P7	208-294kgs	459-648lbs
P8	257-341kgs	568-753lbs
P9	317-386kgs	701-851lbs
P10	373-453kgs	823-1000lbs

**Please note that the above charts are just a guideline, not a rule. Because of different body types, shapes and height we recommend using the hand check method to ensure that the pressure setting is optimal and that the patient isn't bottoming out.*

Operational Guidelines

- Carefully plug power cord of Control Unit into a properly grounded power source. Turn on the master mechanical power switch on the right side panel. The STANDBY LED will illuminate on the keypad.
- Press the Main Power Button. The OPERATE LED will illuminate and the airflow will commence.
- The Rhythm Multi mattress utilizes true low air loss therapy (1275L per minute) at all times during operation. Set optimal air pressure for the patient by using a hand check method.
- The mattress defaults in STATIC mode. Press the MODE button once to change the therapy to PULSATE. This setting supplies pulsation therapy where the pressure in the mattress lowers for 90 seconds and then rises back to the original pressure for 90 seconds, a wave-like motion.
- Press the MODE button a second time for the ALTERNATE mode. This setting provides 3 in 1 alternating pressure therapy for treatment and prevention of pressure ulcers. The cells inflate and deflate in a 3:1 cycle, meaning 2/3rds of the body is always supported by air at any one time.
- The CYCLE button controls the alternating pressure cycle time in the ALTERNATE mode. CYCLE is selectable from 3 to 95 minutes A CYCLE marks the time each 1/3 set of bladders are deflated.
- PLEASE REMEMBER TO CLEAN THE FILTER ON THE BACK OF THE RHYTHM MULTI AT LEAST EVERY 30 DAYS. The black filter that is housed in the back of the Control Unit can be rinsed under running water and set to dry for 15 minutes. If you notice any dust clogs on the filter opening, use a vacuum to remove.
- CPR Deflation: The air hose connectors must be disconnected from the controller to quick release the air when in an emergency where CPR is to be performed.

Cleaning

The mattress should be cleaned on the bed weekly using a damp soft cloth and mild detergent. If top cover or base cover becomes grossly soiled, put on clean gloves, plastic gown and eye protection before removing top and base covers and dispose of according to standard hospital procedures for contaminated waste and replace with clean covers.

Covers can be washed and thermally disinfected in a washing machine. Never use phenol based cleaning solutions.